



Bordered SNS77733-3"x3" SNS77744-4"x4"

Non-Bordered SNS74502-2"x2" SNS74520-4"x5"

## Instructions for Use

# Silicone Foam Wound Dressing

Simpurity<sup>™</sup> Silicone Foam Wound Dressings contain a highly absorbent foam pad and gentle adhesive silicone contact layer, available with or without an adhesive border. These dressings help maintain a moist wound healing environment with no irritation upon removal, minimizes trauma to the wound and reduces pain during dressing changes.

#### **Application:**

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin

2. Choose the right size of dressing to cover the wound according to wound area

#### Bordered Silicone Foam:

- 1. Remove liners from the back side of the dressing
- 2. Remove one of the back layers and apply that dressing half to the wound. Then remove the second liner and secure the rest of the dressing to the wound

#### Non-Bordered Silicone Foam:

- 1. Remove liners from the back of the dressing
- 2. Apply the dressing to the wound bed
- 3. Secure the dressing with medical tape or a transparent dressing

### **Changing Frequency:**

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

#### **Dressing Removal:**

Gently remove the tape, if used. Peel from the edge of the used foam dressing to remove it from the wound bed. Repeat application steps.

#### Indications:

Simpurity<sup>™</sup> Silicone Foam dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure ulcers, leg ulcers, non-infected diabetic foot ulcers, donor sites, post-operative wounds, lacerations and abrasions.

#### **Contraindications:**

The product is not indicated for third-degree burns or surgical implantations.