

Instructions for Use

Bordered
SNS77111-3"x3"
SNS77133-5"x5"

Non-Bordered
SNS74422-2"x2"
SNS74445-4"x5"
SNS74466-6"x6"
SNS74477-7"x7"

Silver Silicone Foam Wound Dressing

Simpurity™ Silver Silicone Foam Wound Dressing is a polyurethane foam containing silver ions and a gentle adhesive silicone contact layer, available with or without an adhesive border. This dressing protects the wounds from bacterial penetration while reducing bioburden in the wound bed. Controlled release of ionic silver provides sustained antibacterial effect. Provides longer wear times and pain-free removal.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area

Bordered Silver Silicone Foam:

1. Remove liners from the back side of the dressing
2. Remove one of the back layers and apply that dressing half to the wound. Then remove the second liner and secure the rest of the dressing to the wound

Non-Bordered Silver Silicone Foam:

1. Remove liners from the back of the dressing
2. Apply the dressing to the wound bed
3. Secure the dressing with medical tape or a transparent dressing

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

Dressing Removal:

Gently remove the tape, if used. Peel from the edge of the used foam dressing to remove it from the wound bed. Repeat application steps.

Indications:

Simpurity™ Silver Silicone Foam dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure ulcers, leg ulcers, Diabetic foot ulcers, second degree burns, donor sites, post-operative wounds and skin abrasions.

Contraindications:

The product is not indicated for third-degree burns or surgical implantations.