SafenSimple

afensimple

SafenSimple

Skin Barrier Film

50 Individual Protective Wipe

Instructions for Safe n Simple Skin Barrier Film

Leaves a protective barrier on the skin to help protect the skin from dressing tapes and damage caused by adhesives.

General Instructions for Use:

- 1. Open Package: tear open top or side edge of individual package
- 2. Remove wipe
- 3. Discard packaging
- 4. Gently wipe skin to apply thin protective barrier

5. Allow skin to dry completely before applying tapes, adhesives, or ostomy pouching system

6. Apply pouching system

NOTE: For optimum effectiveness, use an Adhesive Remover before the Barrier Film. Reapply Skin Barrier with every pouching system change

CAUTION: FOR EXTERNAL USE ONLY

Ingredients: Isopropyl alcohol, Butyl ester of PVM/MA, Acetyl Tributyl Citrate.



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172 samples@sns-medical.com · www.SnS-Medical.com Like us on Facebook: https://www.facebook.com/Safe-n-Simple-878501788888756/