


## Instructions for Safe n Simple Skin Barrier Ring - Conforming Seal

Creates a secure and comfortable seal around the stoma to protect the skin from leakage and irritation. Ideal for an Ileostomy or a Urostomy.

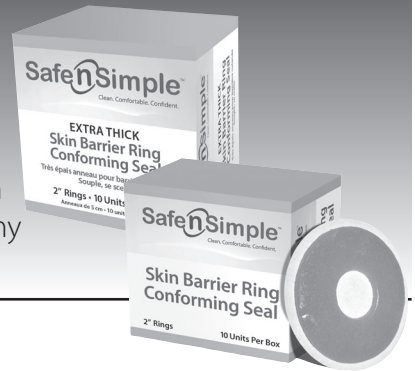

### **Directions for Application:**

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams, and/or soap residues

1. Remove the release liners from both sides of the seal

### **Conforming seal can be used in two ways:**

- 1. Apply directly on skin:** Use your fingers to stretch and shape the seal to fit around your stoma. Make sure you have a snug or comfortable fit...or
- 2. Apply directly on wafer/skin barrier:** Use your fingers to stretch and shape the seal to fit the desired shape around the hole of your wafer/skin barrier pouching system. Center carefully and press firmly in place

### **Directions for Removal:**

1. Gently peel away from skin, working downward and pressing against the surrounding skin

**NOTE:** Conforming seals can be remolded into any shape and used as caulk to fill in uneven skin areas

**Ingredients:** Polyethylene, Silicone Release Paper, Styrene, polymer with isoprene, Polyisobutylene, Gelatin, Pectin, Carboxymethylcellulose (cellulose, carboxymethyl, ether, sodium salt), Mineral Oil, unspecified, Synthetic Resin staybelite ester 10E, Zinc Oxide, USP