

# Hydrocolloids

SNS55402-2"x2" SNS55404-4"x4" SNS21149-7"x7"

## Instructions for Use

# Hydrocolloid Wound Dressing

Simpurity<sup>™</sup> Hydrocolloid Wound Dressing is a thin hydrocolloid sheet that accelerates the healing process by providing an ideal environment for formation of granulation tissue. The dressing forms a gel upon absorbing wound exudate providing a moist wound environment, supporting granulation and epithelialization. The gel formation prevents the dressing from adhering to wounds and can be removed without causing pain and without damaging the new tissue.

## **Application:**

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose the right size of dressing to cover the wound according to wound area
- 3. Remove liners from the back of the dressing
- 4. Apply the dressing to the wound bed

### **Changing Frequency:**

Simpurity<sup>™</sup> Hydrocolloids high absorption capacity binds away excess exudate prolonging dressing change intervals. The dressing is slightly transparent enabling wound inspection. It is recommended to change this dressing three (3) times a week.

#### Indications:

Simpurity<sup>™</sup> Hydrocolloid Wound Dressings are indicated for the management of light to moderately exuding wounds such as: second-degree burns, leg ulcers, pressure ulcers, chronic slow healing wounds and abrasions and can be used as a blister treatment.

#### **Contraindications:**

The product is not indicated for third-degree burns, on exposed muscle, bones or tendons and infected wounds.