

Polymer Highly Absorbent

SNS59020-4"x5" SNS59080-8"x10"

Instructions for Use

Polymer Wound Dressing-Highly Absorbent

Simpurity™ Polymer Wound Dressings are composed of a soft and highly absorbent polymer pad with a white mesh contact layer and light blue adhesive free outer cloth protection layer. Simpurity™ Polymer Foam absorbs the exudate and locks it into the super-absorbent core, allowing for significantly more fluid to be handled by the dressing while keeping it away from the wound and surrounding tissue. This can help to reduce maceration, as well as reduce exposure to harmful components of wound fluid. Use this dressing as a primary or secondary cover dressing.

Application:

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose the right size of dressing to cover the wound according to wound area
- 3. Apply the dressing to the wound (apply the white side to the wound bed)
- 4. Secure the dressing with medical tape

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. Initially it may be necessary to change the dressing every 24 hours. When the exudate reduces then dressing change frequency may slow down.

Dressing Removal:

Gently remove the tape. Peel from the edge of the used dressing to remove it from the wound bed. Repeat application steps.

Indications:

Simpurity™ Polymer dressings are indicated for the management of moderate to heavily exuding wounds such as: surgical incisions, laceratons, abrasions, burns, donor or skin graft sites, or any exudating wound.