

Item# SNS21605 - 4" x 4" 5 sheets/box


Item# SNS21615 - 4" x 4" 5 sheets/box (contains Aloe and Zinc Oxide)



Perfect for red weeping irritated peristomal skin, this thin hydrocolloid sheet can be placed directly on the skin around the stoma to create a smooth platform for the pouching system. The top waterproof layer gives a dry place for your pouching system to adhere to. This Skin Barrier Sheet is gentle and will not impede newly healing skin.

The Skin Barrier Sheet Plus contains all the benefits of the hydrocolloid sheet with Aloe and Zinc Oxide added for extra healing properties. You can apply your wafer or tapes directly over the Skin Barrier Sheet. Can be used along with the crusting technique.

Underneath the Appliance:

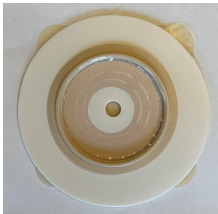


The Skin Barrier Sheet can be placed directly on the skin around the stoma to create a smooth platform for your pouch.

The Skin Barrier Sheet protects the skin from trauma caused by stoma output or by repetitive removal of tapes and adhesives. You can apply your wafer or tapes directly over the Skin Barrier Sheet.

Instructions for use:

1. Prepare the skin and make sure it is clean and dry, using a Peri-Stoma Cleanser & Adhesive Remover wipe.
2. We suggest using a skin barrier wipe or spray to protect your skin and let it completely dry.
3. Cut a hole to the necessary size, then remove the white paper backing from the Skin Barrier Sheet.
4. Place the skin barrier sheet directly on the skin around the stoma.
5. Use your fingertips to smooth and press the skin barrier sheet onto the skin so the skin barrier sheet and skin form a secure seal.
6. Apply the ostomy appliance.

Over the Appliance:

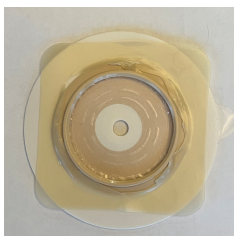

Skin Barrier Sheet can be placed around the wafer to secure the edges from lifting up, creasing or loosening.

Skin Barrier Sheet provides an excellent skin friendly alternative to waterproof tapes to give you a waterproof seal around the wafer when bathing or showering.

Instructions for use:

1. Prepare the skin and make sure it is clean and dry, using a Peri-Stoma Cleanser & Adhesive Remover wipe.
2. Cut a hole to the necessary size, then remove the white paper backing from the Skin Barrier Sheet.
3. Place the skin barrier sheet directly around the edges of the wafer or appliance forming a picture frame around the border of the wafer.
4. Use your fingertips to smooth and press the skin barrier sheet around the appliance or wafer to create a secure waterproof seal.