

Xeroform Impregnated Gauze

SNS58818-1"x8" SNS58822-2"x2" SNS58844-4"x4" SNS58859-5"x9"

Instructions for Use

Xeroform Impregnated Gauze Wound Dressing

Simpurity™ Xeroform Impregnated Gauze Wound Dressing is a sterile, occlusive, non-adhering dressing consisting of an absorbent, fine-mesh gauze impregnated with an antimicrobial petrolatum blend. The dressing will conform to all body contours.

Application:

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose the right size of dressing to cover the wound according to wound area
- 3. Open the package to expose the saturated dressing
- 4. Apply the dressing to the wound bed
- 5. Secure with a secondary cover dressing

Changing Frequency:

Follow your medical professionals directions for dressing change frequency. Typically, the dressing should be changed daily.

Indications:

Simpurity™ Xeroform Impregnated Gauze is indicated for the management of minimal to no-exudate wounds such as: Pressure sores, leg ulcers, first and second degree burns, donor sites, deep wounds, and wounds with necrosis and slough, surgical wounds with light exudate where mild medication and deodorization are desired.

Contraindications:

The product is not indicated for use on wounds with heavy exudate.