

Alginate

Instructions for Use

Alginate Wound Dressing

SNS50702-2"x2" SNS50704-4"x4" SNS50720-4"x5" SNS50732-4"x8" SNS50764-8"x8" SNS50712-12" Rope

Simpurity[™] Alginate Dressing is made of natural fibers derived from seaweed through a series of special processes, which design it to be highly absorbent. The dressing absorbs exudate and forms a gel-like covering over the wound, maintaining a moist environment for wound healing. The gel-like surface also keeps the dressing from adhering to the wound for painless removal.

Application:

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose the right size of dressing to cover the wound according to wound area. Use a sterile pair of scissors to cut the dressing or fold it to fit into the wound bed
- 3. Apply the dressing to the wound (either side of this dressing can be applied to the wound bed)
- 4. Cover and secure the dressing with a secondary dressing

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. Typically the dressing should be changed daily.

Dressing Removal:

Gently remove the secondary dressing. Repeat application steps.

Indications:

Simpurity[™] Alginate dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure ulcers, leg ulcers, cavity wounds and can be used post debridement for hemostasis.

Contraindications:

The product is not indicated for full thickness third-degree burns, dry wounds, infected wounds or surgical implantations.