



# Introducing...Assure C Odor Eliminator

## Assure

### Features & Benefits:

- Is a clear formulation that eliminates the fear or concern of staining associated with the blue color deodorant formulations.
- Delivers outstanding odor control with no perfumes or artificial scents added and contains a lubricant to help empty pouch contents.
- Convenient tilt tip dispenser bottle.
- 4 oz. and 8 oz. sizes available



### Product Numbers:

SNS41404 - 4oz. Bottle  
SNS41408 - 8oz. Bottle

## How to Use...

Eliminates odor and lubricates the inside of an ostomy pouch to help make emptying safe and simple.

**For Cut-to-Fit Wafers/Barriers:** Cut the wafer/barrier with the release paper on.

### For Drainable Pouches - Odor Control and Emptying:

1. Apply pouch.
2. Add a small squirt into the bottom of the pouch.
3. Rub sides of pouch together to thoroughly coat the inside of the pouch. Repeat steps 1 and 2 after every emptying.

### For Closed End Pouches - Odor Control:

1. Do not remove the release paper on the wafer/barrier to add product.
2. Add a small squirt into the bottom of the pouch.
3. Rub sides of pouch together to thoroughly coat the inside of the pouch. Remove release paper on wafer/barrier and apply in normal manner. Repeat steps 1 and 2 after every pouch change.



Clarkston, MI 48348 • P: 844.767.6334 • F: 248.698.3454

[www.SnS-Medical.com](http://www.SnS-Medical.com)