

# Foam Ionic Silver

SNS72502-2"x2" SNS72520-4"x5"

#### Instructions for Use

## Foam Wound Dressing - Ionic Silver

Simpurity™ Foam Ionic Silver Wound Dressing is a polyurethane foam dressing containing silver ions that protects wounds from bacterial penetration while reducing the bioburden in a wound bed. It consists of a hydrophilic polyurethane foam substrate and silver compound. Controlled release of ionic silver provides a sustained antibacterial effect. Inhibits a broad-spectrum of infection up to 99.99% for a wide variety of bacteria. Provides longer wear times and pain-free removal.

#### **Application:**

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose the right size of dressing to cover the wound according to wound area
- 3. Apply the dressing to the wound (apply the white foam side to the wound. This dressing has a plastic backing to eliminate strike-thru)
- 4. Cover and secure the dressing with medical tape or a transparent dressing

### **Changing Frequency:**

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

## **Dressing Removal:**

Gently remove the tape. Peel from the edge of the used foam dressing to remove it from the wound bed. Repeat application steps.

#### Indications:

Simpurity™ Foam Ionic Silver dressings are indicated for the management of infected moderate to heavily exuding wounds such as: pressure ulcers, leg ulcers, Diabetic foot ulcers, second degree burns, donor sites, post-operative wounds and skin abrasions.

#### **Contraindications:**

The product is not indicated for third-degree burns or surgical implantations.