

SNS51W02-2"x2"
SNS51W03-3"x3"
SNS51W04-4"x4"
SNS51W24-4"x6"

Instructions for Use

Foam Wound Dressing

Simpurity™ Foam Wound Dressings contain a moisture permeable barrier layer designed to protect against exudate strike-through while acting as a protective barrier to outside wound contaminants. Simpurity Foam dressings are designed to be easily removed from wound sites. The non-adhesive construction of the dressing helps to assure that the skin around the wound perimeter is not damaged.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Apply the dressing to the wound (apply the white foam side to the wound. This dressing has a plastic backing to eliminate strike-thru)
4. Cover and secure the dressing with medical tape or a transparent dressing.

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

Dressing Removal:

Gently remove the tape. Peel from the edge of the used foam dressing to remove it from the wound bed. Repeat application steps.

Indications:

Simpurity™ Foam dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure sores, leg ulcers, cavity wounds and can be used as a protective padding.

Contraindications:

The product is not indicated for third-degree burns or surgical implantations.