

Gauze Sterile Sponge

SNS54444-4"x4"

Instructions for Use

Sterile Gauze Sponge Wound Dressing

Simpurity™ Gauze Sterile Sponges are ideal for general wound cleansing, debriding, minor prepping, covering and packing into wounds. Each packet contains two (2) sponges of 12 ply 100% USP woven cotton.

Application:

- 1. Open package and remove the two (2) sponges
- 2. Clean the wound with sterile cleanser or saline water and wipe with gauze sponge or multiple sponges if needed.
- 3. Dry the surrounding skin with a sterile gauze sponge

Sterile Gauze Sponge Wound Dressing:

This dressing can be used as a primary dressing covering, if used with medical tape to secure the edges in place.

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate.

Indications:

Simpurity™ Sterile Gauze Sponge dressings are indicated for use on all types of wounds for cleaning, prepping and can be used as a primary and secondary wound cover dressing.