

Instructions for Use

Sterile Bordered Gauze Wound Dressing

Simpurity™ Sterile Bordered Gauze Sponges are a non-adherent gauze sponge island dressing with an adhesive border. The sponge will absorb exudate for a cleaner wound bed and the adhesive border protects the wound from outside contaminants. These woven sponges are made of 12 ply 100% USP cotton for use as a primary or secondary dressing.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Remove one of the back layers and apply that dressing half to the wound. Then remove the second liner and secure the rest of the dressing to the wound

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. Initially it may be necessary to change the dressing every 24 hours. When the exudate reduces then dressing change frequency may slow down.

Dressing Removal:

Peel up an edge of the adhesive border and gently pull to remove the old dressing. Repeat application steps.

Indications:

Simpurity™ Sterile Bordered Gauze Sponge dressings are indicated for the management of low to moderate exuding wounds such as: partial and full thickness wounds, acute and chronic wounds, surgical wounds, stage I or II pressure ulcers and skin abrasions.