

To Apply

Cleanse application area around where Catheter Stabilization Device will be placed (Figure A) with alcohol prep pad or as directed by your facility. **Allow area to completely dry.**

A skin barrier may be applied according to package instructions. Allow area to completely dry after application.

Remove center adhesive pad cover (Figure B) and place on desired catheter tube location (Figure C). This may be done while preparation area is drying. Leave ample slack in Foley tubing to avoid irritation.

With catheter now held securely in place by adhesive pad, grasp both tabs and feed single tab through opening in the double tab. (Figure D).

While grasping both tabs, pull them in opposite directions until catheter is snugly captured and push each to down on the mating Velcro base surface (Figure E).

Hold catheter and stabilization device in place, peel off one side of the paper backing and press to adhere to skin. Repeat on the other side (Figure F) making certain all edges are adhered firmly to skin.


At this time tabs and tubing may be repositioned for comfort.

To Remove

Pull both tabs away from base and pull the single tab back through the hole on the double tab.

Slowly peel the Catheter Stabilization Device off the skin.

If desired an alcohol prep pad may be used to help loosen the adhesive.

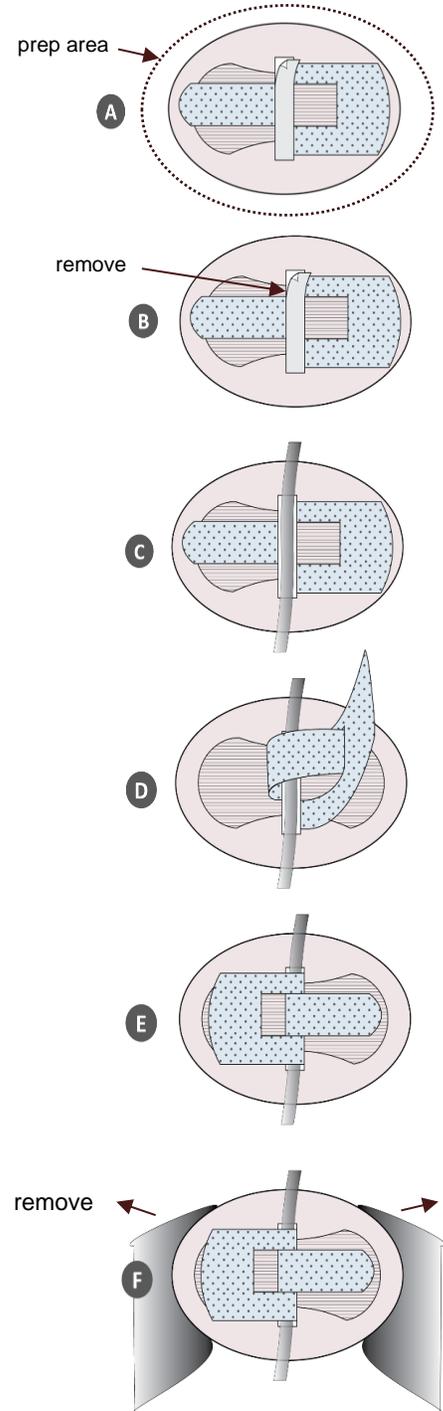

Greystone Medical LLC

Waterford, MI 48327

www.greystone-medical.com +1.248.238.8315