

## Instructions for Safe n Simple Extra Large Adhesive Remover

These large 4"x4.75" soft wipes remove oil, dirt and adhesive residue with a small amount of alcohol to assist in removing more stubborn debris from the skin.



### **General Instructions for Use:**

1. Open single packet
2. Remove wipe
3. Gently wipe skin clean
4. Rinse skin of remaining debris with clean water
5. Make sure to dry the skin before using a skin barrier
6. Used wipe is flushable (not recommended for septic or older plumbing systems)

### **Helps to Remove Pouches:**

1. Gently lift a corner or edge of the wafer, barrier, tape, or hydrocolloid dressing
2. Use the wipe between the skin and the dressing, gently peeling the adhesive back while continuing to wipe
3. Repeat steps 4-6 above

**NOTE:** More than one wipe may be needed, depending on how much residue is on the skin

### **CAUTION: FOR EXTERNAL USE ONLY**

**Ingredients:** Water, Polysorbate 20, Dialzolidenyl urea, Disodium cocoamphodiacetate, Citric Acid, Iodopropynyl Butylcarbamate, isopropanol

**Safe n Simple™**  
Clean. Comfortable. Confident.

Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172  
samples@sns-medical.com · www.SnS-Medical.com

Like us on Facebook: <https://www.facebook.com/Safe-n-Simple>