

## Instructions for Safe n Simple Stoma Caps with Filters

A temporary option to assist with showering, swimming or moments of intimacy. Latex free.



### **Directions for Application:**

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams, and/or soap residues and you have applied your skin barrier film

1. Remove backing paper from the stoma cap and position the cap on the skin around the stoma being sure to get a secure fit

**Colostomy Usage:** A stoma cap works extremely well for those who irrigate and have minimal output from the stoma between irrigations

**Fistula Usage:** For those with a stoma and mucous fistula the cap is an effective dressing to cover fistula, absorb mucous and prevent odor

**Ileostomy Usage:** Cap is not designed to replace normal pouching system. Pad will absorb minimal drainage during short term usage (intimacy, bathing)

### **Directions for Removal:**

1. Use an adhesive remover wipe on the edge of the cap to start to loosen edge continue to peel back from the skin using wipe as you release the cap from the skin