## SafenSimple

1"Wide - SNS20630 2"Wide - SNS21130 1/2 Circle - SNS22222

## <u>Instructions for Safe n Simple</u> Skin Barrier Arc

Protects the skin from trauma caused by repetitive removal of tapes and adhesives



## Apply underneath the wafer/pouch:

Create a smooth platform for your pouch

- 1. Prepare skin by using a Peri-Stomal stoma cleaner and adhesive wipe to provide a clean and dry surface
- 2. A skin barrier wipe or spray is suggested to protect your skin and let it dry completely
- 3. Remove the white paper backing from the skin barrier arc
- 4. Place the skin barrier arc directly on the skin around the stoma
- 5. Use your fingertips to smooth and press the skin barrier arc onto the skin
- 6. Apply the ostomy wafer/pouch

## Apply over the wafer/pouch:

- 1. Apply your wafer/pouch in the normal manner
- 2. Remove the white paper backing from the skin barrier arc
- 3. Place the skin barrier arc directly around the edges of the wafer or pouch forming a picture frame around the border of the wafer. Use 1-3 arcs
- 4. Use your fingertips to smooth and press the skin barrier arc around the edges of the wafer to create a secure waterproof seal

1"Wide - SNS20630 - 30 Arcs/Tray 2"Wide - SNS21130 - 30 Arcs/Tray 1/2 Circle - SNS22222 - 20 Arcs/Tray

Composition: Hydrocolloid



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172 samples@sns-medical.com · www.sns-medical.com Like us on Facebook: https://www.facebook.com/Safe-n-Simple