

Instructions for Safe n Simple Assure C Odor Eliminator

Eliminates odor and lubricates the inside of an ostomy pouch to make emptying safe and simple.



For Drainable Pouches:

1. Apply pouch
2. **Bottle:** add a small squirt into the bottom of the pouch
Packet: add the complete contents of the packet. Close the pouch in the normal manner
3. Rub sides of pouch together to thoroughly coat the inside of the pouch
Repeat steps 1 and 2 after every emptying

For Closed End Pouches:

1. Do not remove the release paper on the wafer/barrier to add product
2. **Bottle:** add a small squirt into the bottom of the pouch. **Packet:** add the complete contents of the packet
3. Rub sides of pouch together to thoroughly coat the inside of the pouch
Remove release paper on wafer/barrier and apply in normal manner

CAUTION: FOR EXTERNAL USE ONLY