

## Instructions for Safe n Simple No Sting Skin Barrier Film

Leaves a protective barrier on the skin to help protect sensitive skin from trauma caused by removal of adhesive from the skin surface.



### **General Instructions for Use:**

1. Open Package: tear open top or side edge of individual package
2. Remove wipe
3. Discard packaging
4. Gently wipe skin surrounding stoma to apply thin protective barrier
5. Allow skin to dry completely before applying tapes, adhesives, or ostomy pouching system
6. Apply pouching system

**NOTE:** For optimum effectiveness, use Peri-Stoma Cleanser & Adhesive Remover before the No-Sting Barrier Film. Reapply Skin Barrier with every pouching system change

### **CAUTION: FOR EXTERNAL USE ONLY**

**Ingredients:** Purified Water, Diglycol/CHDM/Isophthalates/SIP Copolymer, Glycerin, Sorbitol, Octoxynol-9, Dimethicone Copolyol, Propylene Glycol, Methyparaben, Propylparaben, Diazolidinyl Urea