# SafenSimple



## **Directions for Application:**

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams, and/or soap residues

#### 1. Remove the cap from the tube

### Paste can be applied in two different ways:

1. Apply directly on skin: Squeeze the paste to fill any crevices, scars or uneven surfaces and apply in a full circle around the base of the stoma...or 2. Apply directly on wafer/skin barrier: Cut the hole in your wafer/pouch first. Then remove the liners from the back of the wafer/pouch. Squeeze the paste to form a complete circle around the hole of your wafer/skin barrier pouching system

3. Press firmly in place with your fingers

## **Directions for Removal:**

1. Use an adhesive remover wipe to remove any residue

## CAUTION: FOR EXTERNAL USE ONLY



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172 samples@sns-medical.com · www.SnS-Medical.com

Like us on Facebook: https://www.facebook.com