

Instructions for Safe n Simple Adhesive Remover

Effectively removes adhesive residue left from tapes, ostomy wafers and pouches from the skin.



General Instructions for Use:

1. Open single packet
2. Remove wipe
3. Discard open package
4. Gently wipe skin to dissolve adhesive build up
5. Rinse the skin with water to clean off any residue
6. Make sure skin is clean and dry before using a skin barrier

Helps to Remove Pouches:

1. Gently lift a corner or edge of the wafer, barrier, tape, or hydrocolloid dressing
2. Use the wipe between the skin and the dressing, gently peeling the adhesive back while continuing to wipe
3. Repeat steps 4-6 above

NOTE: More than one wipe may be needed, depending on how much adhesive is on the skin

CAUTION: FOR EXTERNAL USE ONLY

Ingredients: Isopropyl alcohol, Isoparaffin, Petroleum Naphtha, Fatty acid isopropyl ester.



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172
samples@sns-medical.com · www.SnS-Medical.com

Like us on Facebook: <https://www.facebook.com/Safe-n-Simple-878501788888756/>