

Instructions for Use

Hydrogel Impregnated Gauze Wound Dressing

Simpurity™ Hydrogel Impregnated Gauze Wound Dressing is designed to provide a moist wound healing environment. The dressing contains a high viscosity gel impregnated into the gauze pad making it easy to apply to the wound bed.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Open the package to expose the saturated dressing
4. Apply the dressing to the wound bed
5. Secure with a secondary cover dressing

Changing Frequency:

The dressing can be changed daily. Follow your medical professionals directions for dressing change frequency.

Indications:

Simpurity™ Hydrogel Impregnated Gauze is indicated for the management of minimal to no-exudate wounds such as: Pressure sores, leg ulcers, first and second degree burns, donor sites, deep wounds, and wounds with necrosis and slough.

Contraindications:

The product is not indicated for use on wounds with heavy exudate.