

Hydrogels

Bordered SNS58312-2"x2" SNS58315-4"x5" Non-Bordered SNS58302-2"x2" SNS58345-4"x5"

Instructions for Use

Hydrogel Absorbent Sheets Wound Dressing

Simpurity™ Hydrogel Absorbent Sheet Wound Dressing can effectively absorb water and wound exudate as well as provide sufficient water content to dry wounds, to maintain a moist environment that promotes wound healing. The product is semi-transparent, so it is easy to observe the healing conditions of the wound. It will not stick to the wound and leaves to residue upon removal.

Application:

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose the right size of dressing to cover the wound according to wound area

Bordered Hydrogels:

- 1. Remove liners from the back side of the dressing
- 2. Apply the dressing to the wound
- 3. Once the dressing is in place, remove the backing films

Non-Bordered Hydrogels:

- 1. Remove liners from the back of the dressing
- 2. Apply the dressing to the wound bed

Changing Frequency:

It is recommended that the non-bordered hydrogel should be changed daily and the bordered dressing is recommended to be changed three (3) times a week.

For infected wounds, the product will not affect general or local drug administration

Indications:

Simpurity™ Hydrogel Absorbent Sheet Wound Dressings are applicable to partial thickness burns, splitting skin, donor sites, and epidermis defects resulting from other causes; leg venous ulcers, pressure ulcers and diabetic ulcers.

Contraindications:

The product is not indicated for full thickness burns and individuals who are hypersensitive to this dressing.