

Super Absorbent Pad Wound Dressing

SNS59022-2"x2" SNS59044-4"x4"

Instructions for Use

Super Absorbent Pad Wound Dressing

Simpurity™ Super Absorbent Pads are made of a non-woven super absorbent fiber that will absorb up to 20x its weight of fluid. One layer of the Super Absorbent Pad is equal to 36 layers of medical gauze. The pads will maintain a moist wound environment by allowing for breathability of moisture and oxygen for wound healing. Anti-seepage to prevent wound from leaking. A great alternative to an ABD pad. Use this dressing as a primary or secondary cover dressing.

Application:

- 1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
- 2. Choose/cut the right size of dressing to cover the wound according to wound area
- 3. Apply the dressing to the wound (apply the white side to the wound bed)
- 4. Secure with a secondary cover dressing

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. Initially it may be necessary to change the dressing every 24 hours. When the exudate reduces then dressing change frequency may slow down.

Dressing Removal:

Gently remove the secondary dressing. Peel from the edge of the used dressing to remove it from the wound bed. Repeat application steps.

Indications:

Simpurity™ Super Absorbent dressings are indicated for the management of moderate to heavily exuding wounds such as: surgical incisions, laceratons, abrasions, burns, donor or skin graft sites, or any exudating wound.