## SafenSimple

## Instructions for Safe n Simple Ostomy Skin Barrier Powder

Protects and promotes healing of irritated or broken skin while absorbing moisture and enhancing comfort



SafenSimple

Ostomy

Skin Barrier Powder

5 oz (148 ml.)

## **Directions for Application:**

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams, and/or soap residues

1. Untwist the cap to open powder

2. **Apply directly on skin:** Dust a light coating of powder onto the affected area, brush off excess

For creating Crusting Technique: Dust the powder onto skin. Use a skin barrier wipe or spray over the top to create a crust to protect traumatized skin 3. Allow to completely dry before adhering appliance

## CAUTION: FOR EXTERNAL USE ONLY

Ingredients: CMC, Pectin, Gelatin, Gum



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172 samples@sns-medical.com · www.SnS-Medical.com Like us on Facebook: https://www.facebook.com/Safe-n-Simple-878501788888756/