

## Instructions for Safe n Simple Peri-Stoma Cleanser & Adhesive Remover

These large soft wipes remove oil, dirt and adhesive residue from the skin so the ostomy pouch will adhere well to the skin.



### **General Instructions for Use:**

1. Peel back the label slowly or open single packet
2. Remove wipe
3. Reseal large package to keep wipes moist
4. Gently wipe skin clean
5. No need to rinse skin
6. Make sure skin is dry before using a skin barrier
7. Used wipe is flushable (not recommended for septic or older plumbing systems)

### **Helps to Remove Pouches:**

1. Gently lift a corner or edge of the wafer, barrier, tape, or hydrocolloid dressing
2. Use the wipe between the skin and the dressing, gently peeling the adhesive back while continuing to wipe
3. Repeat steps 4-7 above

**NOTE:** More than one wipe may be needed, depending on how much residue is on the skin

### **CAUTION: FOR EXTERNAL USE ONLY**

**Ingredients:** Water, Polysorbate 20, Dialzolidineyl urea, Disodium cocoamphodiacetate, Citric Acid, Iodopropynyl Butylcarbamate



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172  
samples@sns-medical.com · www.SnS-Medical.com  
Like us on Facebook: <https://www.facebook.com/Safe-n-Simple>