



Silver Alginate

SNS51702-2"x2"
SNS51704-4"x4"
SNS51720-4"x5"
SNS51764-8"x8"
SNS51712-12" Rope

Instructions for Use

Silver Alginate Wound Dressing

Simpurity™ Silver Alginate Dressing is made of natural fibers derived from seaweed through a series of special processes, which design it to be highly absorbent. The dressing absorbs exudate and forms a gel-like covering over the wound, maintaining a moist environment for wound healing. The gel-like surface also keeps the dressing from adhering to the wound for painless removal. Simpurity™ Silver Alginate can restrain growing bacteria and reduce infection. The dressing has been tested in-vitro and found to be effective against bacteria such as: Staphylococcus aureus, Enterococcus faecalis, Streptococcus pyogenes, Escherichia coli, Pseudomonas aeruginosa, Klebsiella pneumoniae.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area.
Use a sterile pair of scissors to cut the dressing or fold it to fit into the wound bed
3. Apply the dressing to the wound (either side of this dressing can be applied to the wound bed)
4. Cover and secure the dressing with a secondary dressing

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. Typically the dressing should be changed daily.

Dressing Removal:

Gently remove the secondary dressing. Repeat application steps.

Indications:

Simpurity™ Silver Alginate Wound Dressings are intended for infected moderate to heavily exuding wounds that are partial to full thickness including: trauma wounds, pressure ulcers, diabetic ulcers, leg ulcers, graft and donor sites and post-operative surgical wounds.

Contraindications:

Do not use Simpurity™ Silver Alginate Dressing on patients with known sensitivity to silver. Not intended for use on third-degree burns.