

Instructions for Use

Microroll Gauze Wound Dressing

Simpurity™ Microroll Gauze is made of prewashed, 100% woven gauze that has been fluffed to provide for a more bulky cushion to protect wound areas. The 6-ply construction of the dressing provides for excellent absorbency, superior breathability and fast-wicking action. The gauze is infused with PHMB which provides a broad range of antiseptic effectiveness against gram-positive and gram-negative microorganisms and drug resistant strains including MRSA.

Application:

1. Open package and remove the roll
2. Apply to the cleaned wound. Wrap around wound area and secure end by tucking, taping or tying the end.

Microroll Gauze Wound Dressing:

This dressing can be used as a primary or secondary dressing covering.

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate.

Indications:

Simpurity™ Microroll Gauze wound dressings are indicated™ for use on all types of wounds where a wrap dressing is needed.