

## Instructions for Use

### Alginate Wound Dressing

Simpurity™ Alginate Dressing is made of natural fibers derived from seaweed through a series of special processes, which design it to be highly absorbent. The dressing absorbs exudate and forms a gel-like covering over the wound, maintaining a moist environment for wound healing. The gel-like surface also keeps the dressing from adhering to the wound for painless removal.

SNS50702-2"x2"  
SNS50704-4"x4"  
SNS50720-4"x5"  
SNS50732-4"x8"  
SNS50764-8"x8"  
SNS50712-12" Rope

#### **Application:**

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area.  
Use a sterile pair of scissors to cut the dressing or fold it to fit into the wound bed
3. Apply the dressing to the wound (either side of this dressing can be applied to the wound bed)
4. Cover and secure the dressing with a secondary dressing

#### **Changing Frequency:**

Dressing change frequency will depend on the patient's condition and the level of exudate. Typically the dressing should be changed daily.

#### **Dressing Removal:**

Gently remove the secondary dressing. Repeat application steps.

#### **Indications:**

Simpurity™ Alginate dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure ulcers, leg ulcers, cavity wounds and can be used post debridement for hemostasis.

#### **Contraindications:**

The product is not indicated for full thickness third-degree burns, dry wounds, infected wounds or surgical implantations.