

## Instructions for Use

### **Bordered Foam Wound Dressing**

Simpurity™ Foam Adhesive Border Dressing is composed of a soft and highly absorbent polyurethane foam pad to absorb wound drainage and a transparent adhesive border to secure the dressing in place. The dressing helps to maintain a moist wound environment for healing.

#### **Application:**

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Remove one of the back layers and apply that dressing half to the wound. Then remove the second liner and secure the rest of the dressing to the wound
4. Remove the top liner covers from the top of the dressing

#### **Changing Frequency:**

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

#### **Dressing Removal:**

Peel up an edge of the transparent border and gently pull to remove the old dressing. Repeat application steps.

#### **Indications:**

Simpurity™ Bordered Foam dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure sores, leg ulcers, cavity wounds, lacerations and abrasions.

#### **Contraindications:**

The product is not indicated for third-degree burns or surgical implantations.