

## Instructions for Use

### **Foam Wound Dressing - Ionic Silver**

Simpurity™ Foam Ionic Silver Wound Dressing is a polyurethane foam dressing containing silver ions that protects wounds from bacterial penetration while reducing the bioburden in a wound bed. It consists of a hydrophilic polyurethane foam substrate and silver compound. Controlled release of ionic silver provides a sustained antibacterial effect. Inhibits a broad-spectrum of infection up to 99.99% for a wide variety of bacteria. Provides longer wear times and pain-free removal.

#### **Application:**

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Apply the dressing to the wound (apply the white foam side to the wound. This dressing has a plastic backing to eliminate strike-thru)
4. Cover and secure the dressing with medical tape or a transparent dressing

#### **Changing Frequency:**

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

#### **Dressing Removal:**

Gently remove the tape. Peel from the edge of the used foam dressing to remove it from the wound bed. Repeat application steps.

#### **Indications:**

Simpurity™ Foam Ionic Silver dressings are indicated for the management of infected moderate to heavily exuding wounds such as: pressure ulcers, leg ulcers, Diabetic foot ulcers, second degree burns, donor sites, post-operative wounds and skin abrasions.

#### **Contraindications:**

The product is not indicated for third-degree burns or surgical implantations.