

Instructions for Safe n Simple Ostomy Pouch Deodorant

Fantastic odor control with no perfumes or artificial scents added for discreet protection. The blue colored formula helps to see the liquid inside the pouch.



INSTRUCTIONS FOR USE:

After pouch change and each time pouch is emptied:

Bottle: add 8-12 drops into the bottom of the pouch and close pouch.

Gently rub sides of pouch together to distribute liquid throughout the pouch.

***Note: For one piece closed end pouches:**

Do not remove release paper from back of wafer/barrier until deodorant has been added.

Bottle: add 8-12 drops through pouch opening

Remove release paper from back of wafer/barrier and apply system as usual.
Gently rub sides of pouch together to distribute liquid throughout the pouch.

CAUTION: FOR EXTERNAL USE ONLY



Clarkston, MI 48348 · P: 844.767.6334 · F: 248.575.4172
samples@sns-medical.com · www.SnS-Medical.com

Like us on Facebook: <https://www.facebook.com/Safe-n-Simple-878501788888756/>