

Instructions for Safe n Simple Ostomy Pouches

Standard wear drainable skin barrier with a comfortable fit panel and dual adhesive tapes. Cut-to-fit, clamp closure. Latex free.



Directions for Application:

Make sure the skin surrounding the stoma is clean, dry and free from oils, creams, and/or soap residues and you have applied your skin barrier film and have your paste or seal ready to apply

1. Cut the pouch to fit the size of your stoma
 2. Remove the liners from the center area first and position the pouch on the skin being sure to get a secure fit
- *Note: if you use paste or seals you may want to apply them to the back of the wafer/pouch at this point - positioning around the cut hole**
3. Remove the outer liners and flatten the edges with the tips of your fingers
 4. If you use a deodorant you will tilt the open end of the pouch up and pour the recommended amount into the bottom
 5. Roll up the end of the pouch and use provided clip to secure opening

Directions for Removal:

1. Use an adhesive remover wipe on the edge of the wafer/pouch to start to loosen edge continue to peel back from the skin using wipe as you release the wafer/pouch