

Instructions for Use

Silicone Sacrum Bordered Foam Wound Dressing

Simpurity™ Silicone Sacrum Bordered Foam Dressing is composed of a soft absorbent pad that enhances patient comfort and absorbs excess fluids. The gentle silicone border adheres to the surrounding skin, not the wound bed. Helps to minimize pain and trauma at dressing changes. Minimizes pressure ulcers of the sacral region.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Remove one of the back layers and apply that dressing half to the wound. Then remove the second liner and secure the rest of the dressing to the wound

Changing Frequency:

Dressing change frequency will depend on the patient's condition and the level of exudate. This dressing is recommended to be changed three (3) times a week.

Dressing Removal:

Peel up an edge of the dressing border and gently pull to remove the used dressing. Repeat application steps.

Indications:

Simpurity™ Silicone Sacrum Bordered Foam dressings are indicated for the management of moderate to heavily exuding wounds such as: pressure sores of the sacral area, chronic and acute sacral wounds, lacerations and abrasions on shallow wounds.

Contraindications:

The product is not indicated for third-degree burns or surgical implantations.