

IV Derm
SNS52723-2"x3"

Transparent Film
SNS57245-4"x5"

Instructions for Use

Silicone Transparent Films

Simpurity™ Transparent Film and IV Derm Silicone Wound Dressings provide a waterproof barrier to external contaminants. The dressings are permeable to air and moisture vapors for faster wound healing. The silicone adhesive makes the dressing easy to remove from sensitive skin areas and can be changed without pain or damage to the skin surface. The silicone wound dressings can be used as a primary dressing, or as an alternative to tapes.

Application:

1. Clean the wound with sterile cleanser or saline water and dry the surrounding skin
2. Choose the right size of dressing to cover the wound according to wound area
3. Remove the back liners from the dressing
4. Apply the dressing to the wound

Changing Frequency:

Follow your medical professionals directions for dressing change frequency. Typically, the dressing is recommended to be changed three (3) times a week.

Dressing Removal:

Gently peel from the edge of the used dressing to remove it from the wound bed. Repeat application steps.

Indications:

Simpurity™ Silicone Transparent Film dressings are indicated™ for the management of wounds such as: skin tears, surgical wounds, superficial abrasions, 1st degree burns, stage I and II pressure ulcers, covering of eye orbits during surgeries, protecting IV, feeding tubes and catheter sites.

Contraindications:

Simpurity™ Silicone Transparent Film dressings are not indicated for use on wounds with moderate to heavy exudate.